



## inLOOP LISTENER

inLOOP products are made exclusively in the USA. Some electronic parts are not produced in the US and have to be imported.

### SPECIFICATIONS

Loop receiver tuned to pick up any induction loop signal that meets IEC 60118-4.

Bass/Treble switch to reduce background hum from electronic noise.

### DIMENSIONS

Listener 3.5" x 2.5" x 1"  
Package 3" x 8" x 12"  
2 lbs.

### CONTENTS

inLOOP Listener  
9 volt battery  
Headphones  
Instruction card  
Storage bag

### WARRANTY

The inLOOP Listener is warranted for 5 years; excluding damage or failure caused by misuse, neglect or excessive electric current or voltage fluctuations.

569 South State Street  
Shelby, MI 49455  
info@inLOOP.tv  
(231) 798-2399



**inLOOP**

## INSTRUCTIONS FOR USE

The package contains: a plastic storage pouch, an American made inLOOP Listener, headphones and a 9 volt battery.



Attach the battery to the terminal, place the battery in the compartment and clip the cover in place.

Turn the volume down by rotating the upper left knob counterclockwise. Plug the headphones into the inLOOP Listener to turn the power on. Place the headphones over your ears and adjust the headband for a comfortable fit. Turn the volume up to a comfortable level.



**The power is turned on by plugging the headphones into the headphone outlet. To conserve battery life unplug headphones when not in use.**

If there is a noticeable hum in the sound produced by the inLOOP Listener, depress the "BASS - TREBLE" button. This is also helpful to sharpen the sound; depending on the user's hearing, this will usually make the speech sounds clearer.

In a looped environment, the  listener provides clarity and amplification for people without T-Coil equipped hearing aids.